

# Ez - Master<sup>®</sup> (Porcelain auto-mixing plate) Product No. B-007


## Feature

Ez-Master<sup>®</sup> will make your dental porcelain work much better and easier.  
(Save your time & materials by using Ez-Master<sup>®</sup>)

Porcelain powder will be mixed automatically and will be moisturized continuously on the special filter of Ez-Master<sup>®</sup>. You will use the mixed porcelain powder in the most suitable condition for build-up. Ez-Master<sup>®</sup> will be effective for all dental technicians, from beginners to masters

## Direction for use

1. Place the round Ceramic base (c) into the container.  
(Peel the plastic off of Ceramic base)
2. Pour distilled water or universal modeling fluid over the ceramic base. (Fill 1/3 of the container with water or universal modeling fluid)
3. Put Absorption paper (b) on the Ceramic base (c).
4. Put Special filter (a) on the Absorption paper (b), at this moment, the glossy surface of Special filter must be upper direction.
5. Special filter (a) will be moisturized slowly. Start to work after Special filter (a) is wet completely.

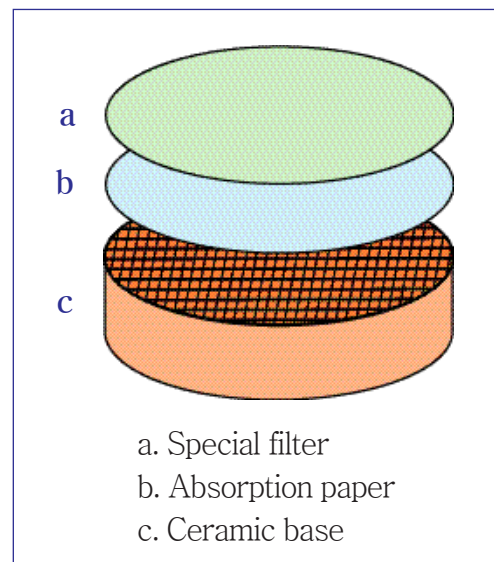

\* Do not use the yellow papers placed between special filters, this yellow papers are protective papers for the special filters.

## Notice

1. Clean up Ez-Master<sup>®</sup> at least once two weeks regularly.
2. Bake Ceramic base at 700 ~ 1200°C in porcelain furnace when the Ceramic base has been polluted by bacteria.
3. Special filter can be reusable for long time. If necessary, it is no obstacle to boil it. However, throw away if Absorption paper has been polluted or torn.
4. Absorption paper (B-AP) and Special filter (B-SF) are refilled.